



# Miracle Dressing™ Post-Discharge Instructions for Home Health Agency Care

## Attention Home Healthcare Provider:

Please be advised that your patient: \_\_\_\_\_ (name) had a Miracle Dressing™ System (Class 1 medical device) applied during their recent facility admission (SNF / Hospital).

This was applied to either treat or prevent a pressure injury on: \_\_\_\_\_ (date applied).

### **DO NOT REMOVE THIS DRESSING!**

The dressing was designed to fall off when all of the Marine Extract has been dissolved through daily washing and moisturizing (usually at least 21 days from application).

The dressing requires daily washing with a mild liquid soap and rinsing using soaked gauze pads (tap water is fine). Moisturizers and medications such as topical antibiotics may be applied directly over the dressing, and will be absorbed.

Please take the time to educate a responsible family member on how to maintain this daily routine. **This is most important to achieve the best results possible.**

Again, please review the daily care video available via the QR code below or online at [www.MBETHHealth.com](http://www.MBETHHealth.com) under the "How To Use" navigation tab. Please share it with a family member as well. Thank you for your time and consideration in caring for this patient.

If you have any questions about daily care, please **DO NOT REMOVE** the dressing, instead contact:

MBET Health at 800-431-6238  
Monday through Friday  
5:00 am to 5:00 pm PST

Referring Facility: \_\_\_\_\_



[MBETHHealth.com/application-clean](http://MBETHHealth.com/application-clean)